

Disposable Child's Face Mask



KEY FEATURES

The 3-layer design provides BFE and PFE >98%
One size fits children from six to fourteen years old (14.0x 9.0cm)
Adjustable nose clip significantly reduces air penetration
Ultrasonically welded earloops stay firmly attached
AZO, PH, formaldehyde & fluorescent brighteners test qualified

MATERIAL COMPOSITION

Outer and inner layers – spunbond Nonwoven polypropylene
Filter – meltblown nonwoven polypropylene
Straps – polyester elastic cord
Nose clip – polypropylene coated galvanized wire
This mask contains no components made from natural rubber latex
Approximate weight of product: 2.5grams / 0.088oz



INTENDED USAGE

The high protective mask is intended to be worn by children to cover the user's nose and mouth
The mask provides a physical barrier to block pollutants from nose and mouth and to reduce risks of spreading viruses and breathing in airborne particles

DO NOT USE FOR

Children protective mask is not a respirator and is not designed to replace N95
This mask is not for a place where the infection risk level through inhalation exposure is high

TIME USE LIMITATION

Replace the mask when it becomes dirty, damaged, or difficult to breathe through

TESTING AND STANDARDS

GB/T 38880-2020 ASTM F2100 Level 3 (report from Intertek)

TEST	GB/T 38880 2020	ASTM F2100 Level 3	Typical Value
Bacterial Filtration Efficiency, %	≥95	≥98	99
Differential Pressure, mm H ₂ O/cm ²	N/A	< 6.0 (59 Pa/cm ²)	4.2 (41 Pa/cm ²)
Particulate Filtration Efficiency 0.1 Micron, %	≥90	≥98	99
Synthetic Blood Resistance mmHg	Not Required	160 (21.3 kPa)	160 (21.3 kPa)
Flame Resistance	Burn < 5s	Class 1	Pass

SHELF LIFE AND STORAGE

2 years from the date of manufacture
Use by date on product certificate in MM/YYYY format
Store masks in a dry, ventilated and non-corrosive environment, away from fire sources and inflammables
Avoid heavy pressure, direct sunlight, rain and snow during transportation

Warnings

It is not recommended to wear the mask for children when feels difficult breathing, if necessary, please wear by following the doctor's advice or change other proper respirators.
Discard mask package materials and other items unrelated to mask wearing.
Children should wear the mask under adult supervision, and caretakers should observe and teach children to wear the mask correctly.
Children should not engage in rough and tumble play or exercise at or above moderate intensity while wearing masks, if encounter discomfort, skin allergy or other symptoms during wearing, please take off the mask immediately and consult a doctor if necessary.
Keep the masks dry, avoid getting wet during usage and replace a new one if necessary.
It is not recommended to be reused and cleaned.
It is not intended for exchange after usage.

ORDERING INFORMATION

DESCRIPTION	Case Size	Case Weight	Masks/Box	Boxes/Case	Masks/Case
Children Mask	52.5*37.5*42.5cm	8.4kg	50	40	2000
	52*46.8*36cm	12kg	50	60	3000