

# 9500-N95S



## KEY FEATURES

- NIOSH approved N95 rating
- FDA 510(k) exemption
- 4 layers design, a smaller size for 9500-N95
- Adjustable nose clip significantly reduces air penetration
- Ultrasonically welded adjustable head bands
- Comfortable urethane foam nosepiece to reduce leakage

## MATERIAL COMPOSITION

- Outer and inner layers - polyethylene
- Filter (two layers)- melt-blown nonwoven polypropylene
- Straps - polyester elastic cord
- Nose clip - aluminum
- Nose pad - urethane foam
- This mask contains no components made from natural rubber latex
- Approximate weight of product: 8.0grams / 0.28 oz



## INTENDED USAGE

Protective mask is intended to be worn by general public or healthcare personnel and covers the user's nose and mouth. The mask is used for protection against particles, such as those from minerals, coal, iron, ore, flour and certain other substances as well as liquid or non-oil based particles from sprays that do not also emit oil aerosols or vapors.

## DO NOT USE FOR

- The mask is not for gases, vapors and oil aerosols.
- The mask is not for particulate concentration exceeding 10X PEL/OEL.
- The respirator is not guaranteed to protect against the risk of contracting a disease or infection.

## TIME USE LIMITATION

Replace the mask when it becomes dirty, damaged, or difficult to breathe through.

## TESTING AND STANDARDS

- NIOSH Approved N95 Particulate Respirator
- Meets NIOSH 42 CFR 84 N95 requirements for a minimum 95% filtration efficiency
- NIOSH Approval Number: TC-84A-5463
- FDA 510(K) Exemption

## SHELF LIFE AND STORAGE

- 3 years from the date of manufacture
- Use by date on box in MM/YYYY format
- Store masks in the original packaging, away from contaminated areas, dust, sunlight, extreme temperatures, excessive moisture, and damaging chemicals
- Store in temperatures between -4°F (-20°C) and +86°F (+30°C) and not exceeding 80% humidity

## Warnings

The masks are not intended for reuse and shall be discarded after each procedure. Check your mask's production date. Masks expire three years after the production date, after which they may not provide full protection. Always follow the instructions for use to ensure mask is properly sealed. Not recommended for children under 3.

## ORDERING INFORMATION

DESCRIPTION	Case Size	Case Weight	Masks/Box	Boxes/Case	Masks/Case
9500 N95S	36 x 31 x 41.5cm	3.6kg	20	12	240